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## Material

**Stabil:** bomull, polyester, elastan. **Comfort:** bomull, polyamid, polyester, elastan.

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220701:4

# Instruction for use – Thumb braces


Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users CMC1- and MP-joint of the thumb after injury, surgery, chronic pain, osteoarthritis or overuse problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

## Application

1. Open up all hook and loop straps and close the thumb strap in corresponding circumference of your thumb. Place your thumb in the brace as illustrated in picture 1.
2. Close all hook and loop straps and adjust them so that the brace fits comfortably. Picture 2
3. If the brace has a splint, if needed you can take it out of the splint pocket and bend it to the appropriate shape and angle of the users hands and needs. Place it back in the splint pocket.

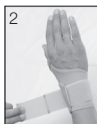Cut off excess hook and loop straps for a cleaner and neater appearance. Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

## Material

**Stabil:** cotton, polyester, elasthan. **Comfort:** cotton, polyamide, polyester, elasthan.

**Neoprene:** synthetic rubber, polyester, polyamide. **Soft:** polyester, viscose, elastane, nylon. All materials are free from latex.

## Washing instructions

Close the hook and loop straps and remove the splint, if the brace has one, from the splint pocket. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

Size	*Dimensions	Size	*Dimensions	Size	* Dimensions
XXS	10 - 12 cm	M	16 - 18 cm	XL	20 - 22 cm
XS	12 - 14 cm	L	18 - 20 cm	XXL	22 - 24 cm
S	14 - 16 cm				

\*Measure wrist circumference. Measurements should be seen as guidelines. Single patient use. If the product contains neoprene, use is recommended for maximum 4 hours at a time.

**Recycling:** If the product has a removable splint, take it out and sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Västberga Allé 26, SE-126 30 Hägersten, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

220701:4