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### **INTENDED PURPOSE**

*Reducing digit friction between toes, corns and calluses of the skin.*

*Cushion and relieve pressure exerted by footwear.*

*Moisturise the skin, improve the feel and appearance of the skin.*

*Accelerating the healing process of scars and closed wounds.*

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### **PATIENT TARGET GROUP**

*Any person who is at risk of pressure or friction damage to the fingers. Or already have them, as long as the wound is closed.*

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### **INTENDED USERS**

*Intended for use in home and professional healthcare facilities by medically trained and untrained users.*

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### **MODE OF ACTION**

*The tubes are elastic and can be easily adapted to different finger sizes.*

*The gel flows in all directions, limiting the relative movement of the skin when rubbing against a moving surface (shoes, socks, etc.).*

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### INSTRUCTIONS FOR USE

1. *Identify the area of the fingers most at risk of being damaged by pressure or shear/friction forces or fragile, dry skin at risk of damage.*
2. *Select the most appropriate diameter size.*
3. *Clean and dry the detected application area.*
4. *Remove the product from the packaging and retain it for storage when not in use.*
5. *Take the required measurement and make the cut.*
6. *Open the tubular, increasing the diameter of the tube so that it can fit snugly on the finger. The product is elastic, but try to not stretch more than 4 times its size.*
7. *Both ends of the tubular can be applied to the skin indistinctly.*
8. *They can be completely covered by socks, stockings and shoes.*

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### CLEANING

*Designed to be washable and reusable on a single patient.*

*Mildly warm water and soft soap, sprinkle with talcum powder and allow to dry naturally.*

#### **End of life disposal**

*Replace when:*

- « *Visibly worn.*
- « *Becomes difficult to clean.*

*At the end of its service life, clean the product according to point 7 and dispose of it with non-hazardous waste.*

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### **CONTRAINDICATIONS**

*Diabetics and those with poor blood circulation should seek professional advice.*

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### **WARNINGS AND PRECAUTIONS**

*Only for use in intact skin, do not use on open or weeping wounds.*

*For external use only.*

*Single patient use.*

*Do not use more than 16 hours per day.*

*Inspect the skin at regular intervals.*

*Discontinue use and seek healthcare professional advice if:*

- « *Loss of skin integrity occurs i.e., breakdown.*
- « *Deep tissue injury is suspected or confirmed.*
- « *Any skin irritation is present.*
- « *Heat rash or reddening of the skin occurs.*

*Before application or reapplication, ensure that the skin is completely dry.*

*Tubes contains a high-grade medical mineral oil which may leach onto absorbent surfaces. Protect non-color fast fabrics.*

*Keep out of reach of children.*