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DEVICE DESCRIPTION

Gel cap to protect toes and nails in-between toes and footwear chafing.

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INTENDED PURPOSE

Reducing digit friction between toes, corns and calluses of the skin.

Cushion and relieve pressure exerted by footwear.

Moisturise the skin, improve the feel and appearance of the skin.

Accelerating the healing process of scars and closed wounds.

The version with tip hole is to allow the skin to breathe.

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PATIENT TARGET GROUP

Any person who is at risk of pressure or friction damage to the fingers. Or already have them, as long as the wound is closed.

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INTENDED USERS

Intended for use in home and professional healthcare facilities by medically trained and untrained users.

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MODE OF ACTION

The caps are elastic and can be easily adapted to different finger sizes.

The gel flows in all directions, limiting the relative movement of the skin when rubbing against a moving surface (shoes, socks, etc.).

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INSTRUCTIONS FOR USE

1. *Identify the area of the fingers most at risk of being damaged by pressure or shear/friction forces or fragile, dry skin at risk of damage.*
2. *Select the most appropriate diameter size and type of fabric.*
3. *Clean and dry the detected application area.*
4. *Remove the product from the packaging and retain it for storage when not in use.*
5. *Take the required measurement and make the cut.*
6. *Open the cap, increasing the diameter of the tube so that it can fit snugly on the finger. The product is elastic, but try to not stretch more than 4 times its size.*
7. *They can be completely covered by socks, stockings and shoes.*

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CLEANING

Designed to be washable and reusable on a single patient.

Mildly warm water and soft soap, sprinkle with talcum powder and allow to dry naturally.

End of life disposal

Replace when:

- « Visibly worn.
- « Becomes difficult to clean.

At the end of its service life, clean the product according to point 7 and dispose of it with non-hazardous waste.

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CONTRAINDICATIONS

Diabetics and those with poor blood circulation should seek professional advice.

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WARNINGS AND PRECAUTIONS

Only for use in intact skin, do not use on open or weeping wounds.

For external use only.

Single patient use.

Do not use more than 16 hours per day.

Inspect the skin at regular intervals.

Discontinue use and seek healthcare professional advice if:

- « Loss of skin integrity occurs i.e., breakdown.
- « Deep tissue injury is suspected or confirmed.
- « Any skin irritation is present.
- « Heat rash or reddening of the skin occurs.

Before application or reapplication, ensure that the skin is completely dry.

Caps contains a high-grade medical mineral oil which may leach onto absorbent surfaces. Protect non-color fast fabrics.

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Biocompatibility tests

Testing has been conducted in accordance with "ISO 10993-1 Biological Evaluation of Medical Devices - Part 1: Evaluation and Testing within a risk management process".

The specific biocompatibility test conducted are the following:

1. *Cytotoxicity.*
2. *Intracutaneous.*
3. *Skin sensitization.*

10.3 Características físicas y químicas *Physical and chemical characteristics*

Color/Colour:	Transparente con tejido color piel/ <i>Transparent with skin color fabric</i>
Aspecto/Aspect:	Sólido/ <i>Solid</i>
Solubilidad/Solubility:	No es soluble en agua/ <i>No solubility in water</i>
Olor/odour:	Aceitoso/ <i>Oily</i>
Punto de fusión/ <i>Melting point:</i>	140°C
Densidad/ <i>Density:</i>	1 kg/dm ³
Dureza/ <i>Hardness (Shore A):</i>	Debajo 0/ <i>Below 0</i>
Estiramiento/ <i>Stretching:</i>	Superior a 400%/ <i>More tan 400%</i>