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Instruction for use – Triggerfix

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace supports the users finger after injury, surgery, inflammatory conditions, chronic disease or overuse problems.

Application

1. Adjust the aluminiumbar in the brace to the user's finger (proximal phalanx) and desired position.
Picture 1
2. Place the brace on the finger. Picture 2
3. Close hook and loop strap so that the brace fits comfortably. Remember not to tighten too much.

Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affect blood circulation.

Material

Nylon, polyester, aluminiumbar. Free from latex.

Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Separate wash is recommended. Use a washing pouch. Let the brace drip dry.

Size

Size	*Dimensions
S	5 - 6,5 cm
M	6,5 - 8 cm
L	8 - 9 cm

*Measure proximal phalanx circumference. Measurements should be seen as guidelines.

Bilateral.

Single patient use.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB
Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

230416:3