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# Instruction for use – Catell Tape

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Treatment with tape may be contraindicated in certain conditions e.g. ongoing infection, tumor disease, deep vein thrombosis and peripheral neuropathy- in case of uncertainty consult a doctor. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

Kinesio tape could for example be used for relieving, ease of pain, stabilizing or in purpose to reduce swelling. Use on clean and dry skin. Apply with tiny or no stretch. Stretch out the muscle or joint before application. With any signs of redness, itching or other skin conditions, remove the tape and contact the supplier of the tape.

## Material

Cotton, spandex, acrylic glue. Free from latex.

## Size

Roll 5 cm x 5 m  
Single patient use.

## Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.