

## Indications

Use to help prevent finger contractures and skin break down in the palm due to conditions involving abnormal tone and/or weakness of the muscles. Made of closed-cell foam, lined with synthetic fleece-like material. Not made with natural rubber latex.

## Instructions For Use

1. Unhook the hook and loop strap. Place the palm protector on the hand by slipping the thumb hole over the thumb, as shown.
2. Once the protector is on the hand, pull the loop strap across the back of the hand and secure in place with the hook tab. The loop strap may be cut to the desired length.
3. To help further extend the fingers and open up a tight grip, place cylindrical foam or wash cloth between the palm and finger pieces of the protector.



### Note:

Be sure the hook tab is not positioned against the skin. If it is, the protector is placed on the hand inside out.

## Care Instructions

Although the Palm Protectors may be machine or hand washed in warm water and a mild detergent, it is best to wash them by hand to ensure a long life. Let the protectors air dry. They can be machine dried but mild shrinkage may occur.

## Order Information


One adjustable size fits all.

NC52250 Left (3)

NC52251 Right (3)



 North Coast Medical, Inc.  
780 Jarvis Drive, Suite 100  
Morgan Hill, CA 95037 – U.S.A.

 MedPass International Ltd.  
Windsor House, Bretforton,  
Evesham, Worcs.  
WR11 7JJ – UK



Made in Mexico